

Band-aid

一、 Product name, model and specification

Product name : Band-aid

型号 Model specification : 5~200mmx5~200mm

二、 Name, address, production address, contact information and after-sales service unit of the registered person/manufacturer

- 1、 Name: Hunan Zhenheyikang Medical Instrument Co.,LTD
- 2、 Address: No.6 Building Jingxiang Energy, No.55 Xiaguang east road, Gaoxin district Xiangtan, Hunan
- 3、 Production address:NO.6 Building, NO.8 Building,5th floor Jingxiang Energy, No.55 Xiaguang east road,Gaoxin district Xiangtan, Hunan
- 4、 Contact number : +86 17711627084
- 5、 After-sales service unit: Hunan Zhenheyikang Medical Instrument Co.,LTD

三、 Production Registration Certificate No: Xiangtan Medical Products Administration
Production Registration NO 20140004

四、 Product Registration Certificate No、 Product Technical Requirements No:
XTFDA Certified 20190015

五、 Product description, intended use

A sheet or rolled band-aid composed of hemostatic cotton pad, medical tape and isolation paper. The hemostatic cotton pad is made of a material that can absorb the exudate. It does not contain components exerting pharmacological, immunological, or metabolic effects. Does not contain the components listed in the appendix. The ingredients should not be absorbed by the human body. Non-sterile provided, single use.

Used for first aid and temporary dressing of superficial wounds such as small wounds, abrasion and cutting wounds.

六、 Use method : Open the bag can be used

七、 Contraindications, precautions, warnings and reminders

Product contraindications: people with skin allergies use cautiously

Single use:do not touch the middle water pad by hand after open the package

The stickiness of the product is lasting, pay attention to timely replacement to maintain the wound hygiene;

Damaged packaging prohibited use, expired products prohibited use

Avoid high temperature, humidity, direct sunlight

八、 Interpretation of graphics, symbols, abbreviations and other contents used in medical device labels



一次性使用



参考使用说明书



包装破损切勿使用

九、 Installation and use instructions or illustrations

Clean and disinfect the wound before using this product; tear open the small package, apply the middle suction pad to the trauma area, then tear the protective film at both ends, and stick the sticky surface to the skin.

十、 Product maintenance and maintenance methods, special storage, transportation conditions, methods

Products should be stored in a non-corrosive gas and well-ventilated room, transported by ordinary transportation means, keep clean and pollution-free, rain and snow immersion.

十一、 Date of production, shelf life or expiration date

Valid for three years, production batch number, production date, expiry date please refer to the small package or qualification label.

十二、 Accessories list, including accessories, appendix, lossitage replacement cycle and replacement method description

This product has n o accessories

十三、 Date and version of preparation or revision of the specification

This specification was revised in May 2023, version 2023.